



## Technique and Use Guide

Bone Shark Disposable Bone Mill technique and use guide.

Spine ENT Orthopedic Maxillofacial



## Technique and Use Guide

Spine ENT Orthopedic Maxillofacial

- Remove all soft tissue from the bone. Cut bone into 1.5 cm or 1/2 inch size pieces
- Peel open the outer packaging
- While gently holding the plunger in place, carefully and slowly remove the bone mill and place on the surgical table until ready for use
- Fill receptacle to approximately 1/2 full. Apply GENTLE pressure to the plunger. Excess pressure will bind the cutting mechanism.
- With the other hand rotate the handle 4 to 5 times clockwise
- Alternate direction 2 to 3 times to expel the milled bone
- The yield will drop into the collection chamber and is now ready for use in the patient
- Remove the top cutting mechanism and expel the product









Lee Medical

30 Vreeland Drive, Suite 8 Skillman, NJ 08558 USA

toll free: 888.422.7717 www.leemedicalnj.com info@leemedicalnj.com